


RAPID ANTIGEN NASAL SWAB INSTRUCTIONS

Follow these instructions carefully.
Use only materials provided in your kit to collect and store your sample.



1 Wash your hands with hand sanitizer or use wipes provided in the bag.

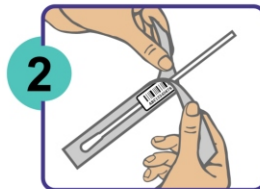

2 Open the swab packaging. Make sure you open from the end opposite of the swab tip.




3 Remove the swab from the packaging, being careful not to touch the soft end, which is the absorbent tip.



4 Insert the entire absorbent tip of the swab into your nostril, until you feel resistance. (The technician will observe to make sure it is done properly.)



5 Slowly rotate 5 times completely, while pressing the tip of the swab gently on the septum of your nose.

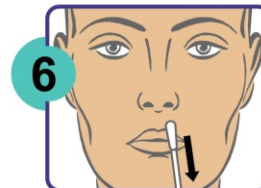


6 Gently remove the swab.




COVID
Diagnostics


covid19-diagnostics.com

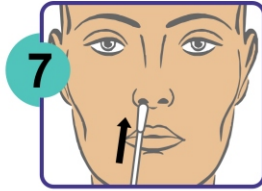

Using the same swab, repeat steps 4-6 in your other nostril.

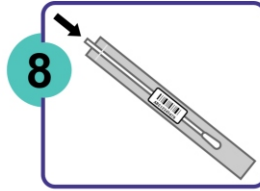

Without touching the soft tip of the swab on any other surface, carefully replace swab back into swab packaging (with barcode), press to seal.

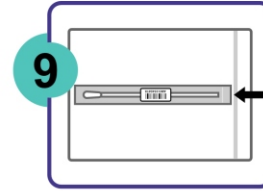

Put completed swab packaging back into ziplock bag and hand kit back to technician for processing.



You will receive an email within a couple hours that your results are available.

WHAT TO DO WHILE YOU WAIT FOR YOUR RESULTS

Whether you are infected or not, wear a face mask, keep a distance of 6 feet or more from others, wash your hands often, and avoid touching your face.

Please keep in mind that SARS-CoV-2 (COVID-19) diagnostic tests are not perfect. No test is. If they were perfect, a positive test would mean that someone carries the virus and a negative test would mean that they do not. With imperfect tests, a negative result means only that a person is less likely to be infected. Thus, we ask that you consider risk factors and individual scenarios and not completely rely on the test results to influence your behavior.